

ASO FL

AS 0050 CH	1 x 50 ml
AS 0100 CH	2 x 50 ml

INTENDED USE

Reagent for quantitative in vitro determination of antistreptolysin O in biological fluids.

SUMMARY OF TEST

Antistreptolysin O (ASO) is an antibody targeted against streptolysin O (SLO), a toxic enzyme produced by group A streptococcus bacteria. These bacteria cause strep throat and a variety of other infections, including skin infections (pyoderma, impetigo, cellulitis). In the course of a streptococcal infection, SLO stimulates the production of specific antistreptolysin antibodies, which act to neutralize the hemolytic properties of the antigen. Measurement of ASO is important in the investigation of post-streptococcal diseases, particularly acute poststreptococcal glomerulonephritis and rheumatic fever.

PRINCIPLE OF THE METHOD

Antibodies of the sample selectively react with Streptolysin O coated to latex, thus producing particle agglutination. The produced turbidity is proportional to the concentration of ASO in the sample, and can be measured at the wavelength of 540 nm.

KIT COMPONENTS

For in vitro diagnostic use only.

The components of the kit are stable until expiration date on the label.

Keep away from direct light sources.

ASO R1 0050: 1 x 40 ml (liquid) white cap
0100: 2 x 40 ml (liquid) white cap

Composition: Buffer pH 8.2, stabilizers and preservatives.

ASO R2 0050: 1 x 10 ml (liquid) red cap
0100: 2 x 10 ml (liquid) red cap

Composition: suspension of latex particles coated with Streptolysin O, stabilizers and preservatives.

Standard: ASO solution - 2 ml

Store all components at 2-8°C.

REAGENT PREPARATION

Use separate reagents ready to use.

Kindly shake R2 vial before use.

Stability: up to expiration date on labels at 2-8°C.

Stability since first opening of vials: use preferably within 60 days at 2-8°C.

PRECAUTIONS

Reagent may contain some non-reactive and preservative components. It is suggested to handle carefully it, avoiding contact with skin and swallow.

Perform the test according to the general "Good Laboratory Practice" (GLP) guidelines.

MATERIALS REQUIRED BUT NOT SUPPLIED

Current laboratory instrumentation. Spectrophotometer UV/VIS with thermostatic cuvette holder. Automatic micro-pipettes. Glass or high quality polystyrene cuvettes. Saline solution.

SPECIMEN

Serum, plasma.

Keep specimens away from direct light sources.

Samples are stable 7 days when stored at 2-8°C and 1 month at -20°C.

TEST PROCEDURE

Wavelength:	540 nm		
Lightpath:	1 cm		
Temperature:	37°C		
dispense:	blank	calibrator	sample
reagent R1	800 µl	800 µl	800 µl
water	10 µl	-	-
calibrator	-	10 µl	-
sample	-	-	10 µl
Mix, incubate at 37°C for 3 minutes.			
dispense:	blank	calibrator	sample
reagent R2	200 µl	200 µl	200 µl
Mix, incubate at 37°C and immediately record absorbances against reagent blank as A ₁ . After exactly 2 minutes, record again absorbances as A ₂ .			

RESULTS CALCULATION

serum/plasma sample:

$$\text{ASO (IU/ml)} = \frac{A_2 - A_1 (\text{sample})}{A_2 - A_1 (\text{standard})} \times \text{standard value}$$

EXPECTED VALUES

Adults < 200 IU/ml

Each laboratory should establish appropriate reference intervals related to its population

QUALITY CONTROL AND CALIBRATION

It is suggested to perform an internal quality control. For this purpose the following human based control sera are available:

MULTINORM CHEMA

with normal or close to normal control values

MULTIPATH CHEMA

with pathological or close to pathological control values.

Please contact Customer Care for further information.

TEST PERFORMANCE

Measure interval

Measure interval depends on the concentration of the highest standard used for calibration.

If such a limit value is exceeded, it is suggested to dilute sample 1+4 with distilled water and to repeat the test, multiplying the result by 5.

Hook effect

No Hook effect is observed with concentrations lower than 4000 IU/ml.

Sensitivity/limit of detection

The limit of detection is 3 IU/ml.

Interferences

No interference was observed by the presence of:

hemoglobin	≤ 1200 mg/dl
bilirubin	≤ 40 mg/dl
lipids	≤ 1000 mg/dl
rheumatoid factor	≤ 800 IU/ml

Precision

intra-assay (n=10)	mean (IU/ml)	SD (IU/ml)	CV%
sample 1	78.0	0.89	1.14
sample 2	190	1.28	0.67

inter-assay (n=10)	mean (IU/ml)	SD (IU/ml)	CV%
sample 1	78.0	1.49	1.91
sample 2	191	3.31	1.73

Methods comparison

A comparison between CHEMA and a commercially available product gave the following results:

$$\begin{aligned} \text{ASO competitor} &= x \\ \text{ASO FL CHEMA} &= y \\ n &= 45 \end{aligned}$$

$$y = 0.95x + 7.7 \text{ IU/ml} \quad r^2 = 0.98$$

WASTE DISPOSAL

This product is made to be used in professional laboratories.

P501: Dispose of contents according to national/international regulations.








REFERENCES

Johnson G.D. *J. Clin. Path.* 1955, 8, 296
Tietz Textbook of Clinical Chemistry, Fourth Edition, Burtis-Ashwood-Bruns (2006), 366.

MANUFACTURER

Chema Diagnostica
Via Campania 2/4
60030 Monsano (AN) - ITALY - EU
phone +39 0731 605064
fax +39 0731 605672
e-mail: mail@chema.com
website: http://www.chema.com

SYMBOLS

	in vitro diagnostic medical device
	batch code
	catalogue number
	temperature limit
	use by date
	caution
	consult instructions for use