# QUANTIPATH CHEMA

5 x 5 ml  
10 x 5 ml

REF QP 0025 CH  
REF QP 0050 CH

LOT 611091  
2020-11

componente component composant componente КОМПОНЕНТ	metodo method méthode méthode метод	valore value valeur valor значение	intervallo range intervalle intervalle диапазон	1SD 1s	unità units unités unidades единица
<b>ALB</b> Albumina Albumin Albumine Альбумин	verde di bromocresolo bromocresol green vert de bromocrésol verde de bromocresol зеленый бромкрезол	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	3.21 2.64 - 3.78 32.1 26.4 - 37.8 488 401 - 575	0.19 1.9 29	g/dl г/дл g/l г/л μmol/l мкмоль/л
<b>ALP</b> Fosfatasi alcalina Alkaline phosphatase Phosphatase alcaline Fosfatasa alcalina Щелочная фосфатаза	IFCC liquida/liquid/liquide/liquido/жидкий	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	225 183 - 267 3.76 3.07 - 4.45	14 0.23	U/l Ед./л μkat/l мккат/л
	DEA/DGKC (valido esclusivamente per reagente Chema) (use for Chema reagent only) (valable uniquement pour le réactif Chema) (sólo válido para el reactivo de Chema) (действительно только для реагента Chema)	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	434 359 - 509 7.25 5.99 - 8.51	25 0.42	U/l Ед./л μkat/l мккат/л
<b>ALT/GPT</b> Alanina aminotransferasi Alanine aminotransferase Alanine aminotransférase Alanina aminotransferasa Аланин-аминотрансфераза	IFCC con/senza piridossalfosfato with/without pyridoxal phosphate avec/sans phosphate de pyridoxal con/sin fosfato de piridoxal с/без пиридоксальфосфатом	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	126 102 - 150 2.10 1.71 - 2.49	8 0.13	U/l Ед./л μkat/l мккат/л
<b>AMY</b> Amilasi Amylase Amylase Амилаза	IFCC/EPG liquida/liquid/liquide/liquido/жидкий	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	196 160 - 232 3.27 2.67 - 3.87	12 0.20	U/l Ед./л μkat/l мккат/л
	CNPG3 (valido esclusivamente per reagente Chema) (use for Chema reagent only) (valable uniquement pour le réactif Chema) (sólo válido para el reactivo de Chema) (действительно только для реагента Chema)	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	187 172 - 202 3.12 2.88 - 3.36	5 0.08	U/l Ед./л μkat/l мккат/л
<b>P-AMY</b> Amilasi pancreatica Pancreatic amylase Amylase pancréatique Amilasa pancreática Панкреатическая амилаза	EPS liquida/liquid/liquide/liquido/жидкий	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	105 87 - 123 1.75 1.42 - 2.08	6 0.11	U/l Ед./л μkat/l мккат/л
<b>AST/GOT</b> Aspartato aminotransferasi Aspartate aminotransferase Aspartate aminotransférase Aspartato aminotransferasa Аспартат-аминотрансфераза	IFCC con/senza piridossalfosfato with/without pyridoxal phosphate avec/sans phosphate de pyridoxal con/sin fosfato de piridoxal с/без пиридоксальфосфатом	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	135 111 - 159 2.25 1.83 - 2.67	8 0.14	U/l Ед./л μkat/l мккат/л
<b>BIL-D</b> Bilirubina diretta Direct bilirubin Bilirubine directe Bilirubina directa Прямой билирубин	Diazo диазо	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	2.76 2.10 - 3.42 47.1 35.7 - 58.5 27.6 21.0 - 34.2	0.22 3.8 2.2	mg/dl мг/дл μmol/l мкмоль/л mg/l мг/л
<b>BIL-T</b> Bilirubina totale Total bilirubin Bilirubine totale Bilirubina total Общий билирубин	DPD	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	4.25 3.47 - 5.03 72.6 59.4 - 85.8 42.5 34.7 - 50.3	0.26 4.4 2.6	mg/dl мг/дл μmol/l мкмоль/л mg/l мг/л
<b>BUN</b> Azoto ureico Urea nitrogen Azote d'urée Nitrogeno ureico Азот мочевины	Ureasi Urease Urease Ureasa Уреаза	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	60.5 51.5 - 69.5 21.6 18.3 - 24.9 0.605 0.515 - 0.695	3.0 1.1 0.030	mg/dl мг/дл mmol/l ммоль/л g/l г/л
<b>CA</b> Calcio Calcium Calcium Calcio Кальций	o-cresolfthaleina complexone o-cresolphthalein complexone o-cresolphthalein complexone o-cresolphthalein complexone о-крезолфталеин комплексон	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	3.51 3.09 - 3.93 7.02 6.18 - 7.86 14.1 12.3 - 15.9	0.14 0.28 0.6	mmol/l ммоль/л mEq/l мЭкв/л mg/dl мг/дл

# QUANTIPATH CHEMA

5 x 5 ml  
10 x 5 ml

REF QP 0025 CH  
REF QP 0050 CH

LOT 611091  
2020-11

componente component composant componente КОМПОНЕНТ	metodo method méthode méthode метод	valore value valeur valor значение	intervallo range intervalle intervalle диапазон	1SD 1s	unità units unités unidades единица
<b>CHE</b> Colinesterasi Cholinesterase Cholinestérase Colinesterasa Холинэстераза	substrato: butirritiocolina substrate: butyrylthiocholine substrat: butyrylthiocholine substrato: butirritiocolina субстрат: бутирилтиохолин	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	5440 4450 - 6430 90.8 74.6 - 107.0	330 5.4	U/l Ед./л μkat/l мккат/л
<b>CHOL</b> Colesterolo totale Cholesterol total Cholestérol total Colesterol total Общий холестерол	CHOD-PAP ID-MS (1)	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	192 162 - 222 4.97 4.22 - 5.72 1.92 1.62 - 2.22	10 0.25 0.10	mg/dl мг/дл mmol/l ммоль/л g/l г/л
		metodo diretto immunologico colorimetrico (valido esclusivamente per reagente Chema) direct immunologic colorimetric method (use for Chema reagent only) méthode colorimétrique immunologique directe (valable uniquement pour le réactif Chema) método colorimétrico inmunológico directo (sólo válido para el reactivo de Chema) колориметрический иммунологический прямой (действительно только для реагента Chema)	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	79.8 64.8 - 94.8 2.08 1.69 - 2.47 0.798 0.648 - 0.948	5.0 0.13 0.050
<b>CHOL-HDL</b> Colesterolo HDL HDL cholesterol Cholestérol HDL Colesterol HDL Холестерол HDL	metodo diretto immunologico colorimetrico (valido esclusivamente per reagente Chema) direct immunologic colorimetric method (use for Chema reagent only) méthode colorimétrique immunologique directe (valable uniquement pour le réactif Chema) método colorimétrico inmunológico directo (sólo válido para el reactivo de Chema) колориметрический иммунологический прямой (действительно только для реагента Chema)	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	99.3 84.0 - 114.6 2.57 2.18 - 2.96 0.993 0.840 - 1.146	5.1 0.13 0.051	mg/dl мг/дл mmol/l ммоль/л g/l г/л
		VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	99.3 84.0 - 114.6 2.57 2.18 - 2.96 0.993 0.840 - 1.146	5.1 0.13 0.051	mg/dl мг/дл mmol/l ммоль/л g/l г/л
<b>CK</b> Creatininas Creatine kinase Créatine kinase Creatina quinasa Креатинкиназа	IFCC liquida/liquid/liquide/liquido/жидкий	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	523 430 - 616 8.73 7.17 - 10.29	31 0.52	U/l Ед./л μkat/l мккат/л
<b>Cl</b> Cloruri Chlorides Chlorures Cloruros Хлориды	ISE potenziometria indiretta ISE indirect potentiometry ISE potentiométrie indirecte ISE potenciometría indirecta Непрямая потенциометрия ISE	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	110 101 - 119 390 354 - 426	3 12	mmol/l ммоль/л mg/dl мг/дл
<b>CREA</b> Creatinina Creatinine Créatinine Creatinina Креатинин	test enzimatico colorimetrico enzymatic colorimetric test test colorimétrique enzymatique test enzimático colorimétrico колориметрический энзиматический тест	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	3.97 3.25 - 4.69 351 288 - 414 39.7 32.5 - 46.9	0.24 21 2.4	mg/dl мг/дл μmol/l мкмоль/л mg/l мг/л
	Jaffé con compensazione compensated compensée compensado Яффе с компенсацией	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	3.85 3.16 - 4.54 340 280 - 400 38.5 31.6 - 45.4	0.23 20 2.3	mg/dl мг/дл μmol/l мкмоль/л mg/l мг/л
<b>CU</b> Rame Copper Cuivre Cobre Медь	Di-Br-PAESA (valido esclusivamente per reagente Chema) (use for Chema reagent only) (valable uniquement pour le réactif Chema) (sólo válido para el reactivo de Chema) (действительно только для реагента Chema)	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	39.3 35.1 - 43.5 250 223 - 277 2.50 2.23 - 2.77	1.4 9 0.09	μmol/l мкмоль/л μg/dl мкг/дл mg/l мг/л
	ferrozina senza deproteinizzazione ferrozine without deproteinization ferrozine sans deproteinisation ferrozina sin desproteinización феррозин без депротеинизации	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	45.6 37.5 - 53.7 255 210 - 300 2.55 2.10 - 3.00	2.7 15 0.15	μmol/l мкмоль/л μg/dl мкг/дл mg/l мг/л

# QUANTIPATH CHEMA

5 x 5 ml  
10 x 5 ml

REF QP 0025 CH  
REF QP 0050 CH

LOT 611091  
2020-11

componente component composant componente компонент	metodo method méthode componente метод	valore value valeur valor значение	intervallo range intervalle intervalo диапазон	1SD 1s	unità units unités unidades единица			
<b>GGT</b> γ-Glutamiltransferasi γ-Glutamyltransferase γ-Glutamyltransférase γ-Glutamiltransferasa γ-Глутамилтрансфераза	liquido std. Szasz liquid std. Szasz liquide std. Szasz liquido std. Szasz жидкий станд. Зейца	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	37°C	205 3.42	169 - 241 2.79 - 4.05	12 0.21	U/l μkat/l	Ед./л мккат/л
	liquido std. IFCC liquid std. IFCC liquide std. IFCC liquido std. IFCC жидкий станд. IFCC		37°C	232 3.87	190 - 274 3.18 - 4.56	14 0.23	U/l μkat/l	Ед./л мккат/л
<b>GLDH</b> Glutamato deidrogenasi Glutamate dehydrogenase Glutamate déshydrogénase Glutamato deshidrogenasa Глутамат дегидрогеназа	DGKC		37°C	43.3 0.723	32.8 - 53.8 0.549 - 0.897	3.5 0.058	U/l μkat/l	Ед./л мккат/л
<b>GLUC</b> Glucosio Glucose Glucose Glucosa Глюкоза	HK G6P-DH / GOD-PAP	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA		256 14.2 2.56	217 - 295 12.1 - 16.3 2.17 - 2.95	13 0.7 0.13	mg/dl mmol/l g/l	мг/дл ммоль/л г/л
<b>HBDH</b> α-Hidroxiacetoaldeid deidrogenasi α-Hydroxybutyrate dehydrogenase α-Hydroxybutyrate déshydrogénase α-Hidroxiacetaldeid deshidrogenasa α-Гидроксибутират дегидрогеназа	DGKC		37°C	331 5.53	271 - 391 4.54 - 6.52	20 0.33	U/l μkat/l	Ед./л мккат/л
<b>K</b> Potassio Potassium Potassium Potasio Калий	ISE potenziometria indiretta ISE indirect potentiometry ISE potentiométrie indirecte ISE potenciometria indirecta Непрямая потенциометрия ISE			6.59 25.8	5.99 - 7.19 23.4 - 28.2	0.20 0.8	mmol/l mg/dl	ммоль/л мг/дл
<b>LACT</b> L-Lattato L-Lactate L-Lactate L-Lactato L-Лактат	metodo enzimatico colorimetrico enzymatic colorimetric method méthode colorimétrique enzymatique método enzimático colorimétrico энзиматический колориметрический метод			33.2 3.69 332	27.2 - 39.2 3.03 - 4.35 272 - 392	2.0 0.22 20	mg/dl mmol/l mg/l	мг/дл ммоль/л мг/л
<b>LDH-L</b> Lattato deidrogenasi Lactate dehydrogenase Lactate déshydrogénase Lactato deshidrogenasa Лактат дегидрогеназа	IFCC liquida/liquid/liquide/liquido/жидкий		37°C	319 5.33	262 - 376 4.37 - 6.29	19 0.32	U/l μkat/l	Ед./л мккат/л
<b>LDH-P</b> Lattato deidrogenasi Lactate dehydrogenase Lactate déshydrogénase Lactato deshidrogenasa Лактат дегидрогеназа	DGKC	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	37°C	602 10.1	494 - 710 8.3 - 11.9	36 0.6	U/l μkat/l	Ед./л мккат/л
<b>Li</b> Litio Lithium Lithium Litio Литий	ISE potenziometria diretta ISE direct potentiometry ISE potentiométrie directe ISE potenciometria directa Прямая потенциометрия ISE			2.03 1.41	1.79 - 2.27 1.23 - 1.59	0.08 0.06	mmol/l mg/dl	ммоль/л мг/дл
<b>LIP</b> Lipasi Lipase Lipase Lipasa Липаза	metodo enzimatico colorimetrico enzymatic colorimetric method méthode colorimétrique enzymatique método enzimático colorimétrico энзиматический колориметрический метод	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA	37°C	108 1.80	90 - 126 1.47 - 2.13	6 0.11	U/l μkat/l	Ед./л мккат/л
<b>MG</b> Magnesio Magnesium Magnésium Magnesio Магний	blu di xilidile xylydyl blue bleu de xylydyle azul de xilidil ксилидил синий	VALORE VALUE VALEUR VALOR ЗНАЧЕНИЕ CHEMA		1.41 2.82 3.43	1.23 - 1.59 2.49 - 3.15 3.01 - 3.85	0.06 0.11 0.14	mmol/l mEq/l mg/dl	ммоль/л мЭкв/л мг/дл

# QUANTIPATH CHEMA

5 x 5 ml  
10 x 5 ml

REF QP 0025 CH  
REF QP 0050 CH

LOT 611091  
2020-11

## Preparación

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**Test procedure**

Use supplied materials as provided.

Additional materials required: automatic micropipette, distilled/deionized water, general laboratory equipment.

**Assay**

Dispense the required volume into a sample cup and analyze in the same way as for the samples. The control should be run daily in parallel with the patient samples and after every calibration. The control intervals should be adapted to each laboratory’s individual requirements. Each laboratory should establish QC procedures that conform with local, state and/or federal regulations or accreditation required.

**Assigned values and ranges**

Target value determination procedure: single assays in 7 external laboratories, with ≥ 3 independent series, under the direction of manufacturer department for external value assignment; this department also performed the evaluation.

The declared value is the median of all values obtained. The corresponding control range is calculated as the target value ±3 SD, with the standard deviation (SD) being the value obtained from several target value determinations.

Determinations were made using the testing reagents and analysis systems available at the time. Results must be within the defined ranges. Each laboratory should establish guidelines for corrective measures to be taken if values fall outside the range.

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## English

### HUMAN BASED UNIVERSAL CONTROL SERUM

#### Intended use

Quantipath Chema is for use in the quality control of general clinical chemistry methods for the quantitative determination of substrates, electrolytes, lipids, enzymes and proteins. The control is used to monitor accuracy or precision both for manual techniques and assays on automated clinical chemistry analyzers.

#### Summary

Quantipath Chema is a lyophilized control serum based on human serum with concentrations / activities found in the pathological range.

#### Kit components

Lyophilized control serum 5 x 5 ml / 10 x 5 ml

#### Reactive lyophilized components

Human serum with chemical additives and tissue extracts of human and animal origin. The origin of biological additives is as follow:

Alkaline phosphatase	placenta (human, recombinant)
ALT/GPT	porcine heart
AST/GOT	human, recombinant
Albumin	human serum
Amylase, total	human saliva / porcine pancreas
Amylase, pancreatic	porcine pancreas
Cholesterol	bovine plasma
Cholinesterase	human serum
Creatine kinase	rabbit muscle
Gamma-GT	human, recombinant
LDH	porcine heart
Lipasi	pancreas (human, recombinant)
GLDH	bacterial, recombinant
Total protein	human serum

The concentrations/activities are lot specific. The exact values and ranges are listed in the enclosed values sheet.

#### Precautions and warnings

For in vitro diagnostic use only. Exercise the normal precautions required for handling all laboratory reagents. This product has been prepared exclusively from the blood of donors tested individually and shown by FDA approved methods to be free from HBsAg and antibodies to HCV and HIV. However as no test method can rule out the potential risk of infection with absolute certainty, the material should be handled just as carefully as a patient sample. In the event of exposure the directive of the responsible health authorities should be followed. (a,b)

#### Disposal

This product is made to be used in professional laboratories. Please consult local regulations for a correct waste disposal. P501: Dispose of contents according to national/international regulations.

#### Handling

Carefully open one bottle of serum, avoiding the loss of lyophilizate, and pipette exactly 5.0 ml of distilled / deionized water. Carefully close the bottle and dissolve the contents completely by occasional gentle swirling within 30 minutes. Avoid the formation of foam. **Important:** with the exception of the **alkaline phosphatase**, all enzymes can be measured immediately. To reactivate the alkaline phosphatase, allow the reconstituted control to stand for one hour at +25°C.

#### Storage and stability

Store at 2-8°C.

Criterion of the stability data specified by the manufacturer: recovery within ±10% of initial value. Stability of lyophilized calibrator serum at 2-8°C: up to the stated expiration date.

STABILITY IN RECONSTITUTED CONTROL SERUM			
	Components	Total bilirubin (protected from light)	Direct bilirubin (protected from light)
15-25°C	12 hours	8 hours	4 hours
2-8°C	5 days	1 day	8 hours
(-15)-(-25)°C	1 month (when frozen once)	2 weeks (when frozen once)	2 weeks (when frozen once)

The possible appearance of a slight green coloration has no effect on the recovery of the values. Store controls tightly capped when not in use.

**Chema**  
D I A G N O S T I C A

Via Campania 2/4  
60030 Monsano (AN)

Italy  
Phone +39 0731 605064  
Fax +39 0731 605672  
e-mail mail@chema.com  
web http://www.chema.com