HbA1c CONTROL SET

2 x 0.5 ml

REF HC S001 CH

LOT -----

componente component composant componente	metodo method méthode método	valore value valeur valor	intervallo range intervalle intervalo	unità unit unidad
LEV 1 HbA1c	metodo immunoturbidimetrico immunoturbidimetric method méthode immunoturbidimétrique método inmunturbidimétrico	---	---	%
LEV 2 HbA1c	metodo immunoturbidimetrico immunoturbidimetric method méthode immunoturbidimétrique método inmunturbidimétrico	---	---	%

Italiano

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Uso previsto

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CONTROL SET

Intended use

HbA1c CONTROL SET is composed of a set of lyophilized materials for quality control of methods for determination of HbA1c in human blood, through immunochemical techniques.

Package components

CONTROL SET (lyophilized) 2 x 0.5 ml

Composition

HbA1c CONTROL SET is a lyophilized material; it is constituted by human blood, containing stabilizers and preservatives.

Assigned values

HbA1c values have been assigned according to NGSP (National Glycohaemoglobin Standardization Program).

Handling

Carefully open each vial of CONTROL SET, avoiding the loss of lyophilizate, and pipette exactly 500 µl of deionized water. Carefully close the bottle and dissolve the content completely by occasional gentle swirling within 30 minutes. Avoid the formation of foam.

Storage and stability

Store at 2-8°C.

The lyophilized product at 2-8°C is stable up to expiration date reported in label. The reconstituted product at 2-8°C is stable 7 days.

Test procedure

Use supplied materials as provided.

Additional materials required: automatic micropipette, distilled/deionized water, general laboratory equipment.

Precautions and warnings

For in vitro diagnostic use only.

Although blood materials used for HbA1c CONTROL SET have been tested and found negative for HBs antigen, antibody to HIV and antibody to HCV, it should be considered potentially infectious and handled in accordance with good laboratory practice using appropriate precautions.

Disposal

This product is made to be used in professional laboratories. Please consult local regulations for a correct waste disposal.

P501: Dispose of contents according to national/international regulations.

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