MICROALBUMIN CALIBRATOR

5 x 1 ml

REF MA CA05 CH

LOT -----

 -----

componente component composant componente	valore di calibrazione / calibration value / valeur de calibration / valor de calibración					unità unit unité unidad
	LEVEL 1	LEVEL 2	LEVEL 3	LEVEL 4	LEVEL 5	
Microalbumina Microalbumin Microalbumine Microalbúmina	----	----	----	----	----	mg/l

Italiano

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Uso previsto

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HUMAN BASED CALIBRATOR SET

Intended use

MICROALBUMIN Calibrator is composed of a set of sera, aimed at the preparation of calibration curves for quantitative determination of Microalbumin in urine, through immunoturbidimetric technique.

Principle of the method

Albumin selectively reacts with an anti-albumin antibody and forms an immunocomplex. The produced turbidity is proportional to the concentration of albumin in the sample and can be measured at the wavelength of 340 nm.

Package components

Calibrators (liquid) 5 x 1 ml

Composition

MICROALBUMIN Calibrator is a liquid product. It is a dilution of defibrinated human plasma with phosphate buffered saline. It also contains stabilizers and sodium azide (< 0.1%) as a biocide.

Assigned values

Microalbumin values have been assigned to ERM-DA470k/IFCC.

Storage and stability

Store at 2-8°C.

The calibrators are ready for use; after vial opening, sera are stable for 6 weeks if stored tightly closed at 2-8°C. Do not freeze.

Test procedure

Use supplied materials as provided.

Additional materials required: automatic micropipette, distilled/deionized water, general laboratory equipment.

Precautions and warnings

For in vitro diagnostic use only.

Exercise the normal precautions required for handling all laboratory reagents. This product has been tested and found free from HBsAg and antibodies to HCV and HIV. However, because no test method can offer complete assurance that products derived from human sources will not transmit infectious agents, it is recommended that this product is handled with the same precautions used for patient specimens.

Disposal

This product is made to be used in professional laboratories. Please consult local regulations for a correct waste disposal.

P501: Dispose of contents according to national/international regulations.

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