RHEUMATOID FACTOR CONTROL SET

2 x 1 ml

REF RF S002 CH

LOT -----

componente component composant componente	metodo method méthode método	valore value valeur valor	intervallo range intervalle intervalo	unità unit unité unidad
Fattore Reumatoide Rheumatoid Factor Facteur rhumatoïde Factor reumatoide	metodo immunoturbidimetrico immunoturbidimetric method méthode immunoturbidimétrique método inmunoturbidimétrico	----	----	IU/ml

Italiano

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Usò previsto

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CONTROL SET

Intended use

RF CONTROL SET contains liquid solutions for quality control of methods for determination of Rheumatoid Factor. The solutions are used to monitor accuracy and/or precision both in automated analyzers and in manual use.

Package components

Controls (liquid) 2 x 1 ml

Composition

RF CONTROL SET is a liquid, defibrinated human serum, containing stabilizers and preservatives.

Assigned values

Rheumatoid factor values have been assigned according to WHO standardization.

Storage and stability

Store at 2-8°C.

The controls are ready for use; after vial opening, sera are stable for 6 weeks if stored tightly closed at 2-8°C. Do not freeze.

Test procedure

Use supplied materials as provided.

Additional materials required: automatic micropipette, general laboratory equipment.

Precautions and warnings

For in vitro diagnostic use only.

Exercise the normal precautions required for handling all laboratory reagents. This product has been prepared exclusively from the blood of donors tested individually and shown by FDA approved methods to be free from HBsAg and antibodies to HCV and HIV. However, because no test method can offer complete assurance that products derived from human sources will not transmit infectious agents, it is recommended that this product is handled with the same precautions used for patient specimens.

Disposal

This product is made to be used in professional laboratories. Please consult local regulations for a correct waste disposal.

P501: Dispose of contents according to national/international regulations.

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