

URIC ACID T FL

AU F100 CH	5 x 20 ml
AU F250 CH	5 x 50 ml
AU F402 CH	4 x 100 ml

INTENDED USE

In vitro diagnostic medical device for quantitative in vitro determination of uric acid in biological fluids (serum) and intended to aid in the diagnosis and to determination of therapy adequacy of gout or kidney diseases. The IVD can be used both in manual or on automatic analyzers. The product is intended for professional use in clinical laboratories.

TEST PRINCIPLE

Uric acid in sample is oxidized to allantoin in presence of the enzyme uricase and H₂O₂ is generated. The H₂O₂ reacts with ADPS and 4-aminoantipyrine in the presence of peroxidase to form a violet dye. The intensity of color formed is proportional to the uric acid concentration and can be measured photometrically to 546 (510 - 560) nm^{2,14-15}.

MATERIALS PROVIDED AND COMPOSITION

UA T R1	F100: 4 x 20 ml (liquid) blue cap
	F250: 4 x 50 ml (liquid) blue cap
	F402: 4 x 80 ml (liquid) blue cap

Composition: Buffer pH 7.0, ADPS ≥ 0.2 mM, stabilizers and preservatives.

UA T R2	F100: 1 x 20 ml (liquid) red cap
	F250: 1 x 50 ml (liquid) red cap
	F402: 1 x 80 ml (liquid) red cap

Composition: Buffer pH 7.7, 4-aminoantipyrine ≥ 1 mM, uricase ≥ 500 U/l, POD > 5000 U/l, stabilizers and preservatives.

Standard: uric acid 5 mg/dl - 5 ml

* Traceability: this method has been standardized against HPLC, according to Original formulation Gindler (1980 - U.S. Patent 4207203) - Weighed in purified material.

MATERIALS REQUIRED BUT NOT SUPPLIED

Current laboratory instrumentation. Spectrophotometer UV/VIS with thermostatic cuvette holder. Automatic micropipettes. Glass or high quality polystyrene cuvettes. Saline solution.

REAGENT PREPARATION

Working reagent: mix 4 parts of reagent R1 with 1 part of reagent R2.

STABILITY AND STORAGE

Stability of working reagent: 15 days at 2-8°C, well capped. Stability of unmixed reagents: up to expiration date on labels at 2-8°C; Stability since first opening of vials of unmixed reagents: preferably within 60 days at 2-8°C.

PRECAUTIONS

UA T R1: Danger. Causes serious eye damage (H318). Wear protective gloves. Eye protection (P280). IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing (P305+P351+P338). Immediately call a doctor (P310).

UA T R2: It is not classified as hazardous.

Standard: It is not classified as hazardous.

N-acetylcysteine (NAC), metazolone and acetaminophen may cause interference in the Trinder reaction¹¹⁻¹³. To avoid interference, the blood withdrawal should be performed before drug administration.

SPECIMEN

Uric acid is not normally affected by additives such as heparin, ethylenediaminetetraacetic acid (EDTA), separation gels, or procoagulants, so the samples should be collected in the same manner routinely used for any laboratory test¹.

Freshly drawn serum are the preferred specimens.

Uric acid is stable 48 hours at 20-25°C, 14 days at 4°C and 4 months at -20°C¹.

TEST PROCEDURE

Wavelength:	546 nm (allowed 510 ÷ 560 nm)		
Lightpath:	1 cm		
Temperature:	37°C		
dispense:	blank	standard	sample
reagent	1 ml	1 ml	1 ml
water	25 µl	-	-
standard	-	25 µl	-
sample	-	-	25 µl

Mix, incubate at 37°C for 5 minutes.
Read absorbances of standard (As) and samples (Ax) against reagent blank.

RESULTS CALCULATION

Serum:

uric acid mg/dl = Ax/As x 5 (standard value)

EXPECTED VALUES

Serum¹³:

Men:	3.5 - 7.2 mg/dl	(0.21 - 0.43 mmol/l)
Women:	2.6 - 6.0 mg/dl	(0.16 - 0.36 mmol/l)

Each laboratory should establish appropriate reference intervals related to its population.

QUALITY CONTROL AND CALIBRATION

Calibration is required with each change in reagent lot number. It is suggested to verify calibration with at least one level of an internal quality control. If control results fall outside acceptable ranges, recalibration may be necessary. For this purpose the following human based control sera are available:

QUANTINORM CHEMA - MULTINORM CHEMA

with normal or close to normal control values

QUANTIPATH CHEMA - MULTIPATH CHEMA

with pathological control values.

If required, a multiparametric, human based calibrator is available:

AUTOCAL H

Please contact Customer Care for further information.

TEST PERFORMANCE

Sensitivity / Limit of Detection (LOD)⁴

The LOD is 0.04 mg/dl.

Analytical specificity:

Interferences⁵

interference does not occur in the presence of:

hemoglobin	≤ 50 mg/dl
bilirubin	≤ 33 mg/dl
Intralipid	≤ 1200 mg/dl

Carry-over effect⁶

BIAS% < 9.81

Accuracy:

Trueness⁶

Total observed error% < 11.97 (allowable total error)

Precision⁷

Repeatability

intra-assay (n=10) mean (mg/dl)	SD (mg/dl)	CV%	
sample 1	5.03	0.02	0.46
sample 2	10.49	0.05	0.49

Reproducibility

inter-assay (n=20) mean (mg/dl)	SD (mg/dl)	CV%	
sample 1	5.03	0.05	0.97
sample 2	10.50	0.11	1.08

Measurement range⁸

0.11 - 30.00 mg/dl

Linearity⁸

the method is linear up to 30 mg/dl.

If the limit value is exceeded, it is suggested to dilute sample 1+9 with saline and to repeat the test, multiplying the result by 10.

Methods comparison⁷

a comparison between Chema and a commercially available product gave the following results:

Serum:

$$\begin{aligned} \text{Uric Acid T FL Chema} &= x \\ \text{Uric Acid AOX FL Chema} &= y \\ n &= 85 \end{aligned}$$

Linear regression

$$y = 1.016x + 0.095 \text{ mg/dl} \quad r = 0.9995$$

Passing-Bablok⁹⁻¹⁰

$$y = 1.018x + 0.081 \text{ mg/dl}$$

Positive and negative Predictive Value

Positive predictive value (PPV): 88.9%

Negative predictive value (NPV): 100.0%

WASTE DISPOSAL

P501: Dispose of contents according to national/international regulations.

NOTICE TO THE USER

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

REFERENCES

1. N. Rifai, A.R. Horvath et al. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, sixth edition 2018.
2. A. Kunst, B. Draeger B. et al. Bergmeyer: Methods of Enzymatic Analysis, third edition 1984.
3. M. Ciaccio, G. Lippi. Biochimica Clinica e Medicina di laboratorio, III edizione 2020, EdiSES Università S.r.l.
4. D.A.Armbruster and T. Pry. Limit of Blank, Limit of Detection and Limit of Quantitation. *Clin Biochem Rev.* 2008; 29(suppl 1): 49-52.
5. M.R. Glick, K.W. Ryder et al. Graphical Comparisons of Interferences in Clinical Chemistry Instrumentation. *Clin. Chem.* 1986; 32: 470-475.
6. E. Theodorsson, B. Magnusson et al. Bias in Clinical Chemistry. *Bioanalysis* 2014; 6(21): 2855-2875.
7. M. Vidali, M. Tronchin et al. Protocollo per la comparazione di due metodi analitici di laboratorio. *Biochimica Clinica* 2016; 40(2): 129-142.
8. CLSI EP0-A:2003 Evaluation of the Linearity of Quantitative Measurement Procedures: A Statistical Approach; Approved Guideline.
9. H. Passing and W. Bablok. A New Biometrical Procedure for Testing the Equality of Measurements from Two Different Analytical Methods. *J. Clin. Chem. Biochem.* 1983; 21: 709-720.
10. L. Bilić-Zulle. Comparison of methods: Passing and Bablok regression. *Biochimica Medica* 2011; 21(1): 49-52.
11. J.R. Genzen, J.J. Hunsaker et al. N-acetylcysteine interference of Trinder-based assays. *Clin Biochem.* 2016; 49(1-2):100-104
12. O. Wiewiorka, Z. Čermáková et al. Drug interference in Trinder reaction. *Euromedlab.* 2017; ISSN 1437-4431
13. D. Barham, P. Trinder. An Improved Colour Reagent for the Determination of Blood Glucose by the Oxidase System. *Analyst.* 1972; 97: 142-145.
14. P. Fossati, L. Prencipe et al. Use of 3, 5-Dichloro-2-hydroxybenzenesulfonic Acid/4-Amino nophenazone Chromogenic System in Direct Enzymic Assay of Uric Acid in Serum and Urine. *Clin. Chem.* 1980; 26(2): 227-231.
15. M. Jelikić-Stankov, P. Djurdjević et al. Determination of uric acid in human serum by an enzymatic method using N-methyl-N-(4-aminophenyl)-3-methoxyaniline reagent. *J. Serb. Chem. Soc.* 2003; 68 (8-9): 691-698.

MANUFACTURER

Chema Diagnostica Srl
Via Campania 2/4
60030 Monsano (AN) - ITALY - EU
phone +39 0731 605064
fax +39 0731 605672
e-mail: mail@chema.com
website: http://www.chema.com

SYMBOLS

Chema Diagnostica uses symbols listed in the ISO 15223-1 (see www.chema.com - Section "Products" for definition of symbols used).

Addition, deletions or changes are indicated with a vertical line on the side of the affected paragraph.