# ADA CALIBRATOR

1 x 1 ml

REF AD S001 CH

LOT -----

 XXXX-XX

componente component composant componente	metodo method méthode método	valore di calibrazione calibration value valeur de calibration valor de calibración	unità unit unité unidad
Adenosina Deaminasi Adenosine Deaminase Adénosine Désaminase Adenosina Desaminasa	cinetico enzimatico enzymatic kinetic cinétique enzymatique cinético enzimático	-----	U/l

## Italiano

### Uso previsto/sommario

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## English

### Intended use/summary

ADA Calibrator is a standard used for the calibration of the "Adenosine Deaminase" assay.

The adenosine deaminase activity has been determined to ensure optimum calibration on automated clinical chemistry analyzers.

### Package components

Calibrator (lyophilized) 1 x 1 ml

### Reactive components in the calibrator

Bovine serum and chemicals additives.

### Calibration values

The calibration values were assigned using enzymatic kinetic methods for the determination of adenosine deaminase. The calibrator value specified is the mean of the values obtained.

### Precautions and warnings

For in vitro diagnostic use only.

Exercise the normal precautions required for handling all laboratory reagents. Product contains highly purified bovine source material from non-BSE (Bovine Spongiform Encephalopathy) countries. Potential infectious agents in the materials have been inactivated. Since no method can offer complete assurance as to the absence of infectious agents, this material should be handled as though capable of transmitting infectious diseases and disposed as biohazard waste or medical waste according to applicable local and national laws.

### Disposal

This product is made to be used in professional laboratories.

Please consult local regulations for a correct waste disposal.

P501: Dispose of contents according to national/international regulations.

### Handling

Carefully open one bottle of calibrator, avoiding the loss of lyophilizate, and pipette exactly 1.0 ml of distilled water. Carefully close the bottle and dissolve the content completely by occasional gentle swirling within 30 minutes. Avoid the formation of foam.

### Storage and stability

Store at 2-8°C.

Stability of lyophilized calibrator serum at 2-8°C: up to the stated expiration date.

After reconstitution: stability is 7 days when stored at 2-8°C.

### Test procedure

Use supplied materials as provided. Additional materials required: automatic micropipette, distilled/deionized water, general laboratory equipment.

### Assay

Dispense the required volume into a sample cup and use as advised.

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