HbA1c CALIBRATOR

5 x 0.5 ml

REF HC CA05 CH

LOT -----

 -----

componente component composant componente	valore di calibrazione / calibration value / valeur de calibration / valor de calibración					unità unit unité unidad
	LEVEL 1	LEVEL 2	LEVEL 3	LEVEL 4	LEVEL 5	
HbA1c	----	----	----	----	----	%

Italiano

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Uso previsto

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HUMAN BASED CALIBRATOR SET

Intended use

HbA1c CALIBRATOR is composed of a set of lyophilized materials, aimed at the preparation of calibration curves for quantitative determination of HbA1c in human blood, through immunochemical techniques.

Package components

Calibrators (lyophilized) 5 x 0.5 ml

Composition

HbA1c CALIBRATOR is a lyophilized material; it is constituted by human blood, containing stabilizers and preservatives.

Assigned values

HbA1c values have been assigned according to NGSP (National Glycohaemoglobin Standardization Program).

Handling

Carefully open each vial of calibrator, avoiding the loss of lyophilizate, and pipette exactly 500 µl of deionized water. Carefully close the bottle and dissolve the content completely by occasional gentle swirling within 30 minutes. Avoid the formation of foam.

Storage and stability

Store at 2-8°C.

The lyophilized product at 2-8°C is stable up to expiration date reported in label. The reconstituted product at 2-8°C is stable 7 days.

Test procedure

Use supplied materials as provided.

Additional materials required: automatic micropipette, distilled/deionized water, general laboratory equipment.

Precautions and warnings

For in vitro diagnostic use only.

Although blood materials used for HbA1c calibrators have been tested and found negative for HBs antigen, antibody to HIV and antibody to HCV, it should be considered potentially infectious and handled in accordance with good laboratory practice using appropriate precautions.

Disposal

This product is made to be used in professional laboratories. Please consult local regulations for a correct waste disposal.

P501: Dispose of contents according to national/international regulations.

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